WIELAND

EXPECT THE DIFFERENCE! BY WIELAND.

- Größte Auswahl an Materialien in höchster Qualität
- Widest range of materials in highest quality
- Grand choix de matériaux. Tous d'excellente qualité
- La più ampia scelta di materiali di primissima qualità
- Gran variedad de materiales de máxima calidad

WIELAND

Gebrauchsanweisung
Instructions for use
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso

ZENOTEC Z¹_{bridge} (translucent)

- Rohlinge zur Herstellung von Zahnersatz
- Blanks for dental restorations
- Disques pour la confection de restaurations dentaires
- Grezzi per la realizzazione di protesi dentali
- Piezas brutas para la confección de prótesis

CE 0483

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Area of application

ZENOTEC Zr_{Bridge} (translucent) are milling discs designed for the production of dental restorations such as crowns, bridges, abutments or primary components.

Safety first

Please read the instructions carefully before you remove the zirconium oxide blank from the packing. They contain important information on working with the ZENOTEC Zr_{Bridge} (translucent) milling discs and serve the interests of your safety and the safety of your patients.

Caution:

Processing ZENOTEC Zr_{Bridge} (translucent) milling discs produces dust which can lead to irritation of the eyes and skin and can damage the lungs. Always ensure that the extraction unit on your milling machine is in good working order. Wear goggles, gloves and a face mask (particle class FFP2).

ZENOTEC Zr_{Bridge} (translucent) milling discs are produced and checked to the highest quality standards. It is absolutely essential to observe the all the processing instructions in order to pass on this quality to the patient. This will avoid any loss of quality of the dental restoration in terms of aesthetics, fit or durability.

General handling instructions

ZENOTEC Zr_{Bridge} (translucent) milling discs are supplied in a partly sintered state. In this state, the material does not yet possess the outstanding properties that it will feature as a finished dental prosthetic restoration. It is therefore necessary to handle the discs with care.

Please check the goods immediately after receipt for:

- Damage to the packaging
- Damage to the product
- The ZENOTEC Quality Code (ZQC) and the name of the manufacturer WIELAND on the packaging

Store ZENOTEC Zr_{Bridge} (translucent) ideally:

- At a constant temperature
- In the original packaging
- In a dry place
- Between 5 °C and 50 °C (40 – 120 °F)

When handling make sure that the ZENOTEC Zr_{Bridge} (translucent) milling discs:

- Is not subjected to knocks or vibrations
- Is not touched with wet hands
- Does not come into contact with any sort of fluid (for example, water, adhesive or inks)
- Is not soiled

If any point in these instructions is not correct, the ZENOTEC Zr_{Bridge} (translucent) milling discs must not be used for the production of dental restorations.

Instructions for dentists

High quality care and subsequent long term patient satisfaction require high-grade materials and professional preparation and processing. You will obtain the best possible patient satisfaction if you follow the recommendations given below:

- Prepare tooth as advised with chamfer or rounded shoulder
- Depth of cut at the preparation margin at least 1 mm
- Remove 1.5 – 2 mm of occlusal/incisal tooth material
- Edge radius: 0.7 mm
- Preparation angle: 6° – 8°
- For laterals no more than two pontics may be positioned between two abutment teeth.

The restoration can be cemented with conventional phosphate or glass ionomer cement. As an adhesive fixing material, WIELAND recommends the following products: RelyX Unicem 3M ESPE or Panavia 21 TC/ Panavia F 2.0 TC from KURARAY. The inner surfaces of the ZENOTEC Zr_{Bridge} (translucent) frameworks should be sandblasted with a max. 50 µm abrasive at a pressure not exceeding 2 bar. For detailed information, please refer to the ZENOTEC Preparation Guide.

- Temporary cementation is contraindicated, since the restoration may be damaged during removal.
- Fixing with resin-modified glass ionomer cements, compomer cements and all cements with a high coefficient of expansion is currently contraindicated, since no long-term empirical studies are available.
- Full contour restorations have to be checked at least twice a year to guarantee that the dynamic occlusion of the tooth structure of the opposing and surrounding natural teeth is in order, and any necessary adjustments should be made accordingly. In doing so, any areas of the occlusal surfaces that have been subjected to adjustment must be conditioned with appropriate polishing tools in order to restore them to maximum surface smoothness.

Contraindications

- In common with all full-ceramic restorations, zirconium oxide frameworks are not suitable for patients suffering from bruxism.
- In cases where space is restricted or the preparation is unsuitable, an alternative form of restoration should be chosen.

Instructions for dental laboratories

When designing crowns, bridges and primary components, the following minimum dimensions should be adhered to:

- Minimum wall thickness for anteriors: 0.4 mm
- Minimum wall thickness for laterals/posteriors: 0.6 mm
- Minimum wall thickness for abutment teeth: 0.6 mm

	*mm ²	**mm
Single crowns	–	0,6
Anterior single crowns connected	5/4	0,6
Posterior single crowns connected	9/9	0,6
Telescope	–	0,5
Anterior bridge 3 units	5/4	0,6
Anterior bridge 4 units	7/6	0,6
Premolar anterior bridge 3 unit	7/9	0,6
Premolar anterior bridge 4 unit	9/12	0,6
Posterior bridge 3 unit	9/12	0,6
Posterior bridge 4 unit (2 premolars)	12/14	0,6
Posterior bridge 4 unit (1 premolar, 1 Molar)	14/16	0,6
Posterior bridge 4 unit (2 Molars)	16/18	0,6

* connection cross section (mm²) OK/UK ** wall thickness (mm)

- Restorations should be designed in such a way as to support the veneering ceramic at the cusp area, so that these can be veneered with an almost uniform layer of ceramic.

Notes on design are given in our "Guidelines for the design and processing of zirconium oxide substructures".

ZENOTEC Color Zr can be used to shade the ZENOTEC Zr_{Bridge}. For monolithic crowns please use the by WIELAND recommended shading liquids.

Milling the discs

If the ZENOTEC Zr_{Bridge} (translucent) milling discs has arrived in an acceptable condition it is then ready to be processed in a ZENOTEC milling machine. The disc diameter is approx. 98 mm so that both large items as well as a number of smaller units may be produced from a single disc. With careful positioning the discs can be processed efficiently.

If any other machines, equipment or tools are used, no guarantee can be given in respect of any frameworks or substructures manufactured in this way.

While processing in one of these machines, ensure that the ZENOTEC Zr_{Bridge} (translucent) milling discs does not come into contact with coolant or compressed air. Please also observe the instructions for your milling machine.

After milling, the ZENOTEC Zr_{Bridge} (translucent) frameworks must be examined carefully and fulfil the following criteria:

- No shiny areas visible on the surface
- No discoloration
- No material broken off
- No visible cracks

If any of these faults are evident, no part of the frameworks may be used to produce dental restorations.

Hard sintering the frameworks

Hard sintering is an essential process in which the ZENOTEC Zr_{Bridge} (translucent) substructure is subjected to a precisely defined temperature treatment. Only after this does the substructure have the necessary strength to guarantee the reliability and performance in clinical application. During hard sintering the substructure shrinks to its final dimensions.

This temperature treatment is ultimately responsible for achieving a perfect fit.

Prior to sintering, all traces of milling dust must be completely removed from the frameworks.

Coloured restorations must be dried slowly and long enough. Please refer to the Instructions of use of the used shading liquids.

The work is hard sintered in the ZENOTEC sinter furnace. The sintering parameters must be set in accordance with the operating instructions for the des ZENOTEC high temperature furnace and must not be modified.

N.B.: A separate sinter program is required for ZENOTEC Zr_{Bridge} (translucent).

The RFID label must not be sintered. Please take it off before placing the blank in the sintering furnace.

Trimming and finishing the frameworks

The hard-sintered ZENOTEC Zr_{Bridge} (translucent) substructure should only be processed further if absolutely necessary. Only water-cooled diamond tools should be used for this or local overheating may arise causing the material to fracture. Please note that you must check your work for defects and cracks before and after finishing with ZENOTEC Light.

Always observe the following rules:

- The thinner the wall thickness the more care is required. The same general guidelines for full ceramic restorations apply – the wall thickness should not be less than 0.6 mm.
- Do not grind with overdue force, only work with minimum pressure.
- Use only diamond tools that are in perfect condition. Reduced cutting performance of the tool generates heat.
- If possible choose tools with a grain size less than 30 µm.
- Avoid grinding the tooth connections and in particular do not grind the basal grooves.
- Avoid sharp edges, try to round off edges.

Veneering of the frameworks

Before veneering, it is recommended to sandblast the frameworks with a 110 – 150 µm abrasive at 2 bar and then rinse with water. For perfect results we recommend veneering the hard-sintered ZENOTEC Zr_{Bridge} (translucent) substructure with ZIROX veneering porcelain from WIELAND. Please follow the instructions for using ZIROX.

Material properties and forms of supply

The typical properties of hard sintered ZENOTEC Zr_{Bridge} (translucent) are set out in the following table.

Density	> 6,0 g/cm ³
Open porosity	0 %
Vickers hardness	1300 HV ₁₀
Crushing strength	2,000 MPa
Bending strength	1200 MPa
E modulus	210 GPa
Fracture toughness	5 MPa · m ^{1/2}
CTE (25 – 500 °C)	10.5 · 10 ⁻⁶ /K
Composition	zirconium oxide (ZrO ₂ + HfO ₂ + Y ₂ O ₃) ≥ 99 % yttrium oxide (Y ₂ O ₃) > 4,5 – ≤ 6,0 % hafnium oxide (HfO ₂) ≤ 5 % aluminium oxide (Al ₂ O ₃) ≤ 0,5 % other oxides ≤ 0,5 %
Type/class	type II/class 6

ZENOTEC Zr_{Bridge} (translucent) is biocompatible.

Available blank sizes ZENOTEC Zr_{Bridge} (translucent)

Nominal diameter (mm)	Nominal thickness (mm)	Ref. No.
98	10	6901301100
98	12	6901301120
98	14	6901301140
98	16	6901301160
98	18	6901301180
98	20	6901301200
98	25	6901301250

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